

Florida Department of Law Enforcement

Alcohol Testing Program

OPERATIONAL PROCEDURES – INTOXILYZER 9000

- ~~All results are reported to three decimal places in g/210L.~~ The instrument must display READY prior to beginning the breath test. Push the green START button to begin the breath test.
 - Enter Breath Test Operator permit number and PIN.
 - DATE/TIME. Verify the displayed date and time. Change if necessary. ~~Enter~~ Verify Breath Test Operator first name, middle initial, ~~and~~ last name, and agency at USER OPERATOR prompts.
 - ~~SWIPE/SCAN OR ENTER DL.~~ Swipe/Scan or manually enter the subject's driver license or identification card and verify ~~DATE/TIME.~~ Verify the displayed date and time. ~~Change if necessary.~~ LAST AGENCY INSPECTION DATE. Verify the Agency inspection date displayed.
 - CYLINDER LOT#. Verify the Dry Gas Standard cylinder lot number and expiration date. Change if necessary.
 - OBSERVATION PERIOD BEGAN. Enter the time the observation period began (must be at least 20 minutes).
 - VIOLATION CODE. Enter the violation code.
 - AIR BLANK. The result must be 0.000.
 - DIAGNOSTICS CHECK. The result must be OK.
 - AIR BLANK. The result must be 0.000.
 - CONTROL TEST. The result must be between 0.075 and 0.085, inclusive.
 - AIR BLANK. The result must be 0.000.
 - PROVIDE SAMPLE NOW. Have the subject provide a breath sample into the instrument.
 - AIR BLANK. The result must be 0.000.
 - PLEASE WAIT. The instrument will countdown the time remaining for the wait period.
 - AIR BLANK. The result must be 0.000.
 - PROVIDE SAMPLE NOW. Have the subject provide a breath sample into the instrument.
 - AIR BLANK. The result must be 0.000.
- Note: If there is no 0.020 g/210L agreement between the first and second breath samples, the instrument will automatically request a third breath sample as follows:
- PLEASE WAIT. The instrument will countdown the time remaining for the wait period.
 - AIR BLANK. The result must be 0.000.
 - PROVIDE SAMPLE NOW. Have the subject provide a breath sample into the instrument.
 - AIR BLANK. The result must be 0.000.
 - CONTROL TEST. The result must be between 0.075 and 0.085, inclusive.
 - AIR BLANK. The result must be 0.000.
 - DIAGNOSTICS CHECK. The result must be OK.
 - AIR BLANK. The result must be 0.000.
 - ~~2-~~ FDLE/ATP Form 38a Breath Alcohol Test Affidavit - Intoxilyzer 9000 will automatically print to the attached external printer containing ~~all the results.~~ ~~3-~~ Complete FDLE/ATP Form 38a Breath Alcohol Test Affidavit - Intoxilyzer 9000.