

# Florida Department of Law Enforcement

## Alcohol Testing Program

---

### OPERATIONAL PROCEDURES – INTOXILYZER 9000

- The instrument must display READY prior to beginning the breath test. Push the green START button to begin the breath test.
  - Enter Breath Test Operator permit number and PIN.
  - DATE/TIME. Verify the displayed date and time. Change if necessary. Verify Breath Test Operator first name, middle initial, last name, and agency at OPERATOR prompt. SCAN OR ENTER DL. Scan or manually enter the subject's driver license or identification card and verify
  - LAST AGENCY INSPECTION DATE. Verify the Agency Inspection date displayed.
  - CYLINDER LOT#. Verify the Dry Gas Standard cylinder lot number and expiration date. Change if necessary.
  - OBSERVATION PERIOD BEGAN. Enter the time the observation period began (must be at least 20 minutes).
  - VIOLATION CODE. Enter the violation code.
  - AIR BLANK. The result must be 0.000.
  - DIAGNOSTICS CHECK. The result must be OK.
  - AIR BLANK. The result must be 0.000.
  - CONTROL TEST. The result must be between 0.075 and 0.085, inclusive.
  - AIR BLANK. The result must be 0.000.
  - PROVIDE SAMPLE NOW. Have the subject provide a breath sample into the instrument.
  - AIR BLANK. The result must be 0.000.
  - PLEASE WAIT. The instrument will countdown the time remaining for the wait period.
  - AIR BLANK. The result must be 0.000.
  - PROVIDE SAMPLE NOW. Have the subject provide a breath sample into the instrument.
  - AIR BLANK. The result must be 0.000.
- Note: If there is no 0.020 g/210L agreement between the first and second breath samples, the instrument will automatically request a third breath sample as follows:
- PLEASE WAIT. The instrument will countdown the time remaining for the wait period.
  - AIR BLANK. The result must be 0.000.
  - PROVIDE SAMPLE NOW. Have the subject provide a breath sample into the instrument.
  - AIR BLANK. The result must be 0.000.
  - CONTROL TEST. The result must be between 0.075 and 0.085, inclusive.
  - AIR BLANK. The result must be 0.000.
  - DIAGNOSTICS CHECK. The result must be OK.
  - AIR BLANK. The result must be 0.000.
  - FDLE/ATP Form 38a Breath Alcohol Test Affidavit - Intoxilyzer 9000 will automatically print to the attached external printer. Complete FDLE/ATP Form 38a Breath Alcohol Test Affidavit - Intoxilyzer 9000.